DIAMOND BRUSHES

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DIAMOND BRUSHES

Instructions for use and safety recommendations for using Diamond Brushes

Daily dental excellence.

A.M. Edelingh Diamond Brushes are for use by the dentist or hygienist for prophylaxis polishing of teeth, they are multi use medical devices. These instruments are class 2A medical devices and are for use in the mouth only by (or under the instruction of) a qualified dental professional. They are manufactured in accordance with the standard ISO 1797 and should only be used in conjunction with a rotary hand piece that conforms to ISO 14457 the operator should ensure that the rotary instrument is correctly installed in the hand piece prior to commencement of any procedure.

Proper use:

- Only use hand piece, angles and turbines that are technically and hygienically flawless, maintained and cleaned.
- Turbine and hand piece must be concentric and true running.
- The instruments must be inserted as deeply as possible into the hand-piece and locked.
- Avoid tilting or levering because of the increased risk of breakage.
- Instruments that are deformed or no longer run true should not be used and must be disposed of.
- Always wear safety goggles.
- Wear a face mask to prevent biological transfer.
- Incorrect use produces poor results and increases the risk.
- These products must only be used by qualified staff.

Safety precautions

These dental instruments were developed and manufactured for their specific prophylaxis polishing application. Incorrect use may harm tissue, cause premature wear, destroy the instrument and endanger the operator, patient or third parties.

Pressure

- Excessive pressure must be avoided at all times.
- Excessive pressure may damage the working sections of rotary polishing instruments or damage the fill material. Heat build up is also increased.

Excessive pressure may cause overheating which injures the pulp or damage brush material. In extreme cases, the instrument may even break, which may cause injury.

Recommended speeds

- Maximum speed 5,000 rpm.
- Keep within the speed range of 1,500 – 5,000 rpm for the best work results and to increase the service life of the instrument.
- If you exceed the maximum permitted speed, the instrument tends to vibrate, which may cause damage to the shaft and/or make the instrument break, with a risk to the user, the patient or third parties. Failure to comply with the maximum permitted speed produces an increased safety risk.

Dispose of worn instruments

- Worn or damaged brushes may cause vibration.
- Bent or non-concentric instruments must also be disposed of.

Worn or blunt rotary instruments induce the user to exert more pressure, which increases the working temperature. This may injure the pulp or tooth.

Storage, disinfection, cleaning and sterilisation.

- Instruments must be disinfected, cleaned and sterilised prior to first use and all subsequent uses on patients. The instrument should be kept in its original packaging at room temperature and protected against dust and moisture until used for the first time.
- Rotary instruments should be kept in hygienic stands, dishes or other suitable containers and they must be protected against dust, moisture and recontamination during storage. If the instruments are not used right away, it is advisable to keep them in their original packaging.
- In general, instruments should not be sterilised at temperatures exceeding 137°C.
- The method of use, reaction time and suitability of disinfectants and cleaning agents for certain types of instruments are covered by the manufacturers' instructions.
- Always follow manufacturers recommended sterilization procedure. Please refer to M+W Dental (steam) sterilisation instructions.

Ensure that the method of disinfection/sterilisation is suitable for the instrument. The relevant details are provided in the documents - Sterilisation instructions - (steam) A.M. Edelingh Diamond Brushes.

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Daily dental excellence.

SCOPE

These instructions are applicable to the processing of A.M. Edelingh Diamond Brushes before first use and each reuse. The instruments are supplied mechanically clean but are not sterile. They should therefore be sterilized before first use and each reuse.

WARNINGS

Used instruments should be considered as contaminated and appropriate handling precautions should be taken during reprocessing. Gloves, eye protection and a mask should be worn. Other measures may be required if there are specific infection or cross-contamination risks from the patient.

LIMITATIONS OF PROCESSING

Reprocessing will have little effect on A.M. Edelingh Diamond Brushes. The end of life is determined by wear and damage in use and the instruments should be inspected for defects during the preliminary cleaning process.

CONTAINMENT AT THE POINT OF USE

Unless there is specific infection or cross-contamination risks, there are no special requirements for containment. The instruments can be transported wet or dry and should be protected from damage. If transported wet there is an increased chance of staining or corrosion. Prolonged storage in disinfectant solutions may result in corrosion and should be avoided.

Delay in reprocessing must be kept to a minimum to avoid contaminants drying thereby making cleaning more difficult.

PREPARATION FOR CLEANING

There are no special requirements unless infection controls require the use of a disinfectant, in which case a disinfectant agent validated for processing of dental rotary instruments must be used and the disinfectant manufacturers' instructions must be followed.

CLEANING

Auto cleaning is the preferred method and should use only validated washer disinfectors and appropriate agents validated for use on dental rotary instruments with the selected machine. Follow the washer disinfectant and the cleaning agent manufacturers' instructions.

If hand cleaning is the only available option, the instruments should be cleaned in a sink reserved for the purpose. Rinse the instruments under running cold water and, keeping them immersed, brush thoroughly away from the body using a neutral cleaning or cleaning/disinfecting agent validated for use on dental rotary instruments. Follow the agent manufacturers' instructions. Care should be taken to avoid spreading contaminants by spraying or splashing during the brushing process. Use wire brushes with caution as brass particles may result in galvanic corrosion and steel particles may cause discolouration.

After cleaning inspect the instrument, with the aid of magnification if necessary, to ensure that all contamination has been removed. Repeat the cleaning process if necessary.

DRYING

Dry the instruments using paper towelling or dry heat not exceeding 140°C.

INSPECTION

Inspect the instruments, with the aid of magnification if necessary, and discard any damaged or corroded instruments.

PACKAGING FOR STERILIZATION

If using a **vacuum autoclave** pack the instrument in dedicated instrument trays or pouches validated for sterilization.

If using a **non-vacuum autoclave** the instruments should not be packed or wrapped but be contained in dedicated bur stands with perforated lids.

NOTE: National legislation may require that instruments are wrapped in pouches for processing in either type of autoclave.

STERILIZATION

Autoclave the instruments for a holding time not less than three minutes at a temperature of between 134°C and 137°C.

The holding time is the minimum time for which the minimum temperature is sustained.

The autoclave manufacturer's instructions must be followed. In particular, care must be taken not to exceed the maximum recommended load for the autoclave.

STORAGE

The instruments should be stored in the sterilization container (bur stand or pouch) until required. Containers or pouches must be dry before opening to avoid recontamination of the contents with water. Storage should be in dry, clean conditions and at ambient temperature.

VALIDATION

These processes have been validated as being capable of preparing A.M. Edelingh Diamond Brushes for reuse. It remains the responsibility of the reprocessor to ensure that the reprocessing as actually performed using the equipment, materials and personnel in the reprocessing facility achieve.